

Application and Fitting Guide

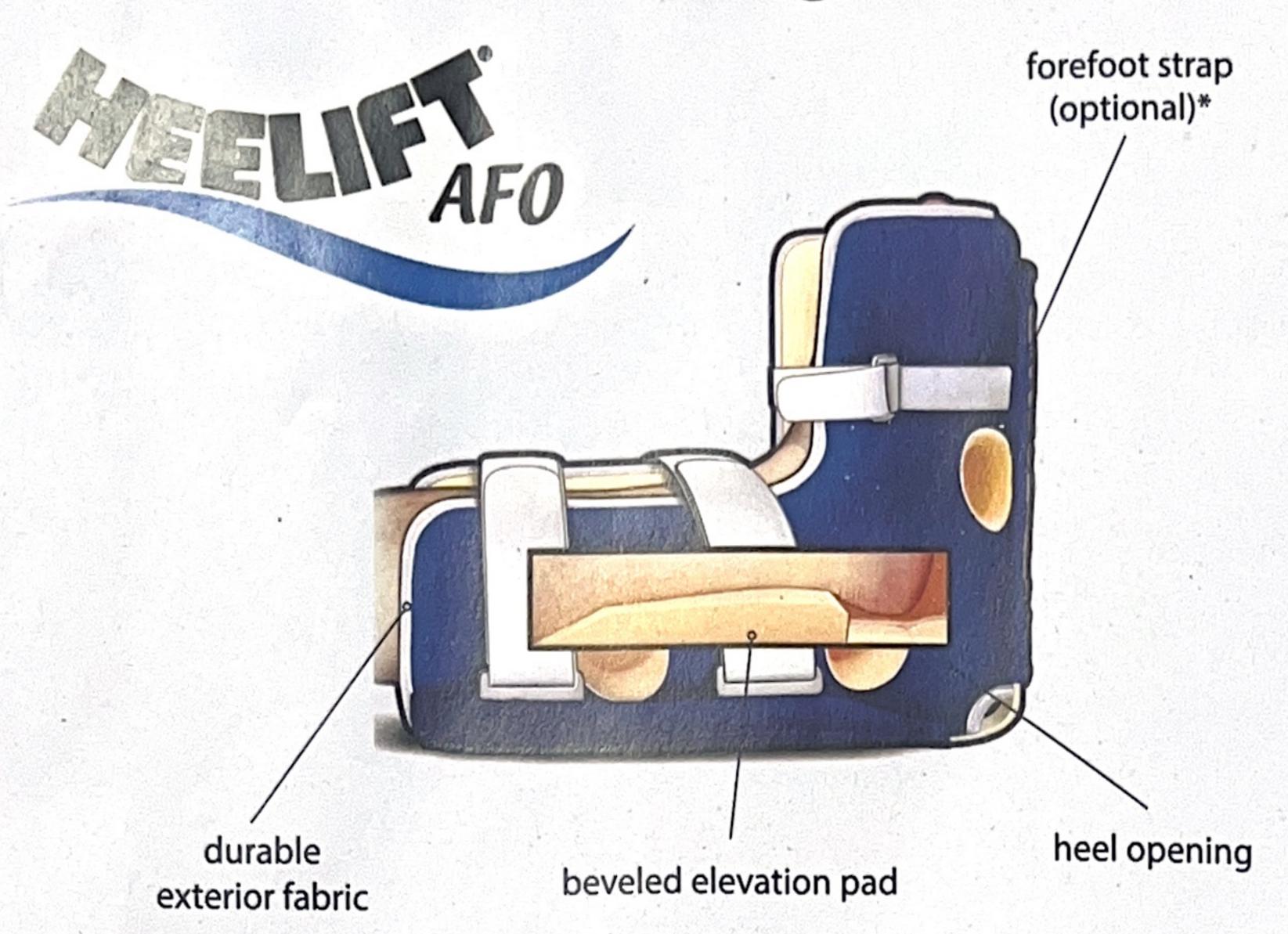

HEELIFT® brand Suspension boots are soft, latex-free foam boots designed to:

- Elevate the heel while lying down
- Protect the foot and ankle from injury due to pressure, friction, and shear forces
- Allow the patient to reposition as needed while HEELIFT® stays in place on the lower limb
- Customizable heel offloading boot which can be accommodated to offload other at-risk areas in addition to the heel in order to prevent secondary ulcers/injuries.

Proper Sizing and Care

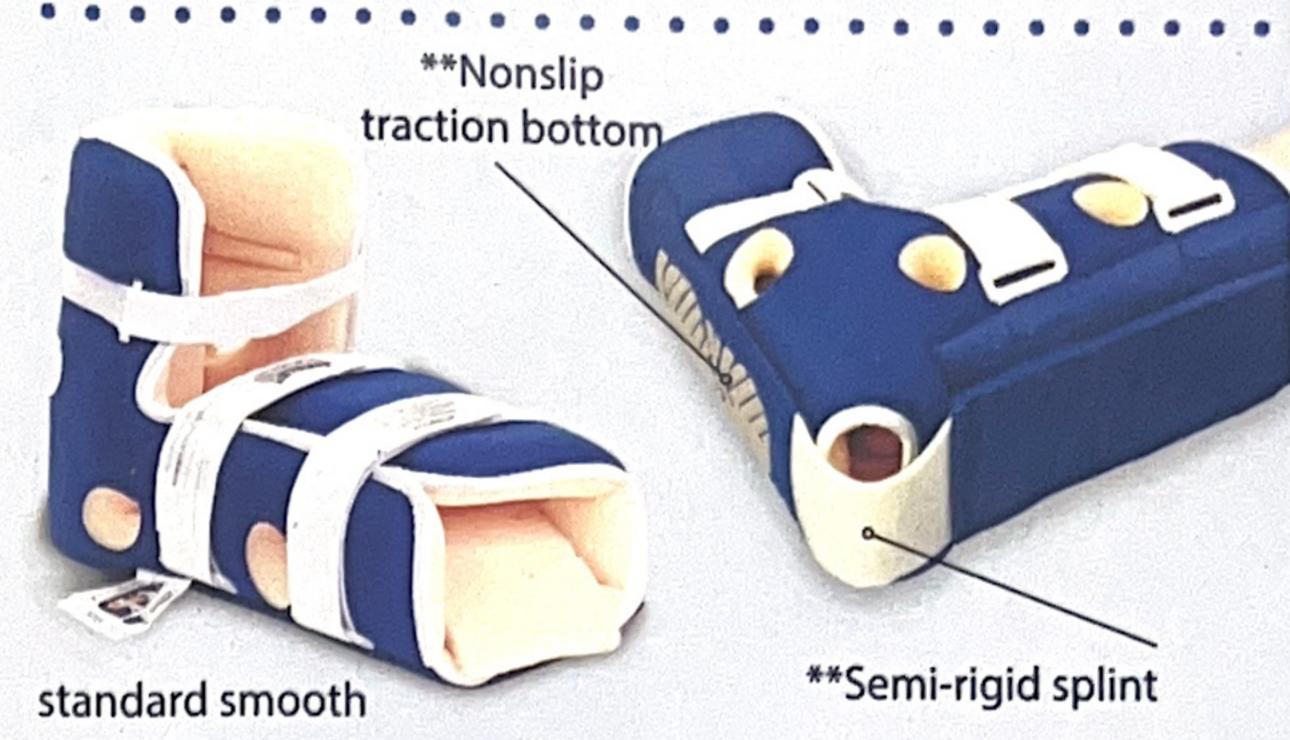

**The built-in splint and non-slip traction sole allows for weight-bearing transfers with assistance.



Standard

10 - 15 in

(25 - 38 cm)

Available with: Smooth or Convoluted interior

✓ Single Patient - Multiple Use





Hand Wash

Machine Dry Low Machine Wash

Sanitize










Secure straps and place boot in laundry bag or pillowcase before machine washing with gentle detergent. Do not use bleach.

*Forefoot strap is optional and should only be used per physician or wound care

specialist discretion. Do not use forefoot strap with any patient with edema or

Easy Application and Fit

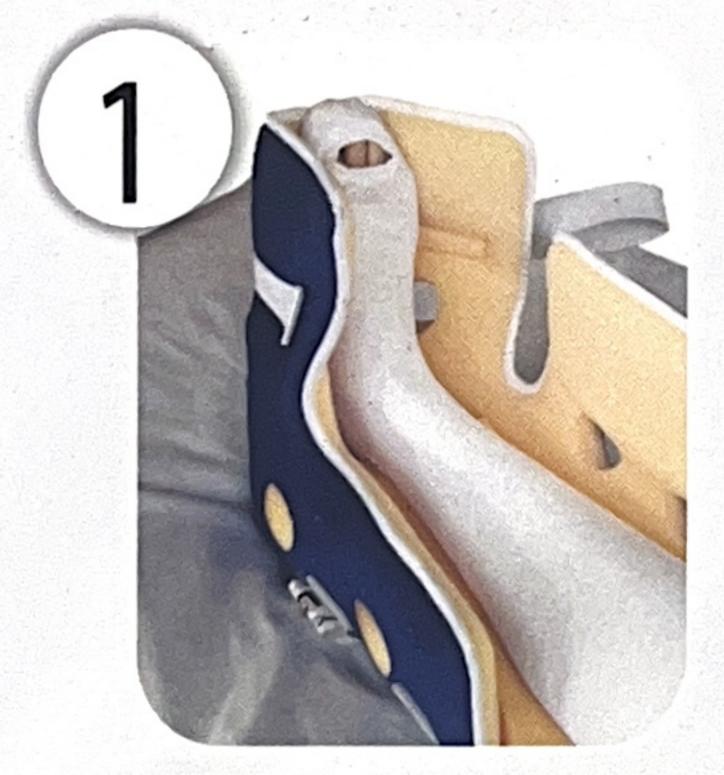

Measure for correct sizing. Place foot inside boot with the heel above the opening.



Thread straps through D-rings. Secure the straps.



with compromised skin integrity.

· Using two fingers, test each strap. Test the fit.

CAUTION: Make sure straps are not in contact with skin.



 Check to confirm heel is off-loaded using heel opening.

Boot should not move on the leg.

WARNING: As per standard of care, remove HEELIFT® boot and inspect skin every 8 hours to ensure no compromise of the skin has occurred. Application and customization of the boot should only be performed per the discretion of the healthcare provider after reviewing IFU cautions and warnings.

Optional Customization



Optional Anti-Rotation Wedge for Foot and/or Hip Rotation (sold separately)



SCD Tubing for DVT Prevention







1316 Sherman Ave. • Evanston, IL 60201 800.254.5438 • 847.328.9540 www.walgreenhealthsolutions.com info@walgreenhealthsolutions.com orders@walgreenhealthsolutions.com

Patent 10,334,905

*Suggested Code - L4396