



CARE PUMP

**MOVE8**  
*Sport line*


Number of chambers	8
Max. pressure [mmHg]	220
Number of proprietary therapeutic pre-programs	6
Smart gradient' function	(mmHg, %, final pressure)
Precise regulation of pressure and time	in steps of 1, 5 or 10
Setting of individual pressure in each chamber	included
Number of independent channels	1
Adjustment of the speed of the treatment	1-5
Cuffs	overlapping system
Display	7" TN lcd color touch screen
Weight [kg]	2,9
Dimensions [mm]	290x130x130
Power supply	battery 21VDC/1A
Estimated battery life from full charge	4 h*
Warranty	1 year

\*test carried out using the "Warm-up, prevention" program and the pressure of 100 mmHg

# Accessories




## Included:

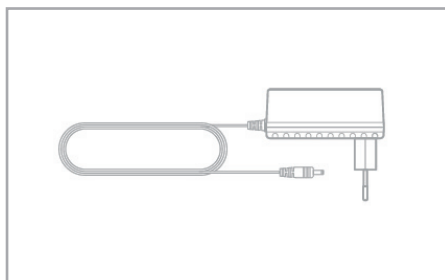


power supply

## Optional:

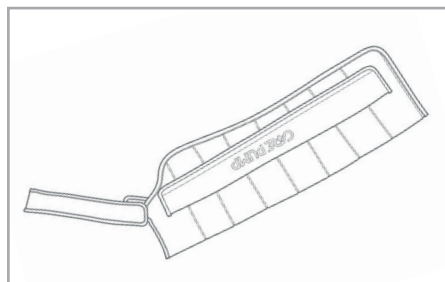


8-chamber arm cuff

CPARM8T

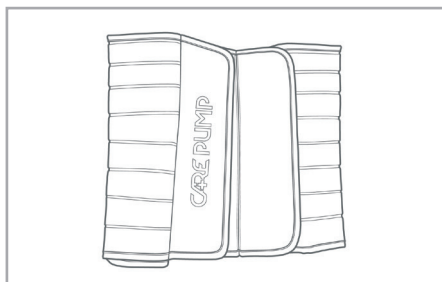


8-chamber waist cuff

CPWAIST8T

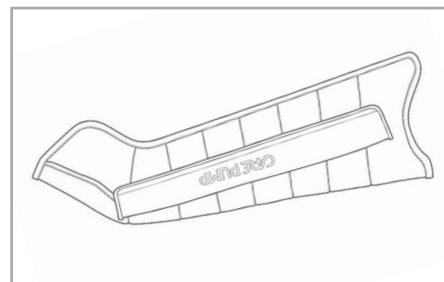


8-chamber left leg cuff

CPLEG8TL

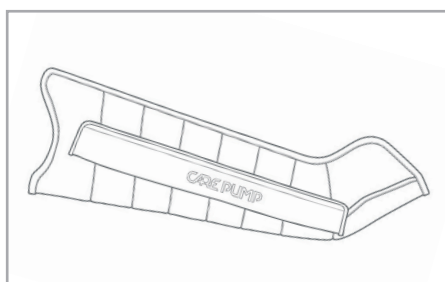


8-chamber right leg cuff

CPLEG8TR

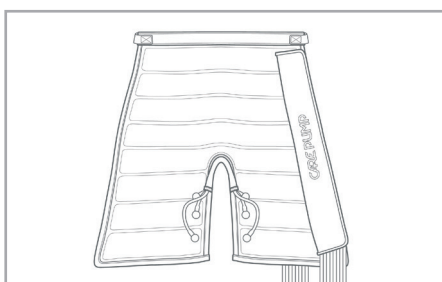


8-chamber short pants

CPSHORTPANTS8T

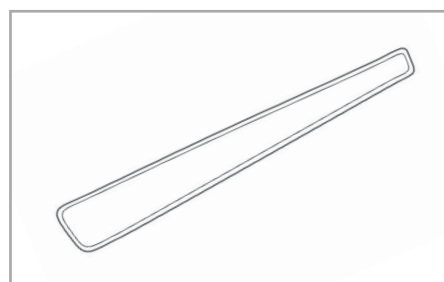


Extension zipper

CPLEGEXT8

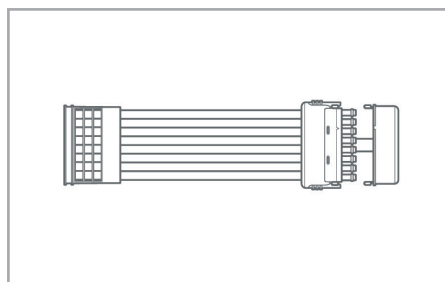


Double splitter

CPDOUBLECONNECTOR8

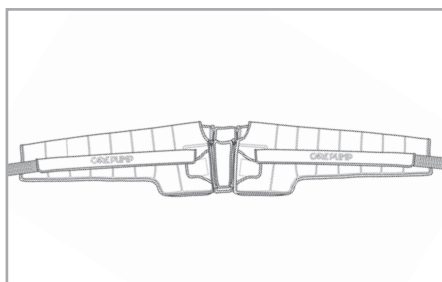


8-chamber double cuff for arms,  
shoulders and chest

CPDOUBLEARM8T

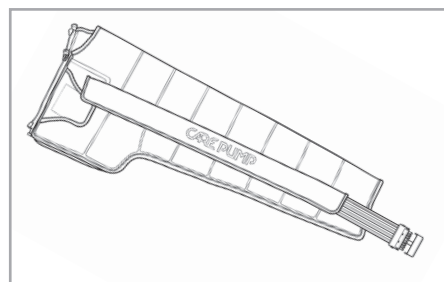


8-chamber left arm, shoulder  
and chest cuff

CPARMSB8TL

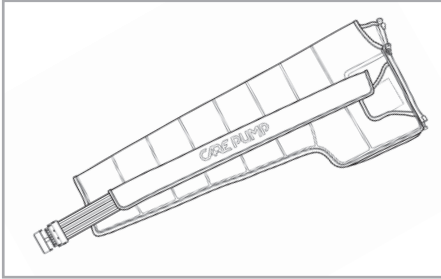


8-chamber right arm, shoulder  
and chest cuff

CPARMSB8TR

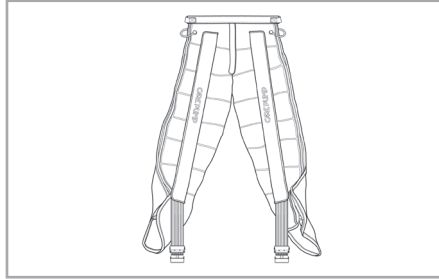


8-chamber full pants

CPLONGPANTS8T

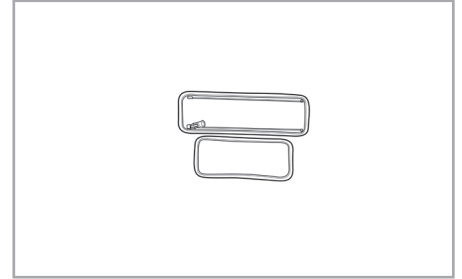


2 x Extension zippers  
for 8-chamber double cuff for  
arms, shoulders and chest

CPDOUBLEARMEXT8

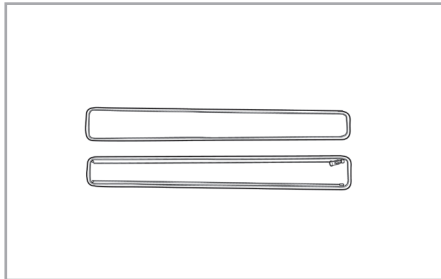


2 x Extension zippers  
for 8-chamber short pants

CPSHORTPANTSEXT8

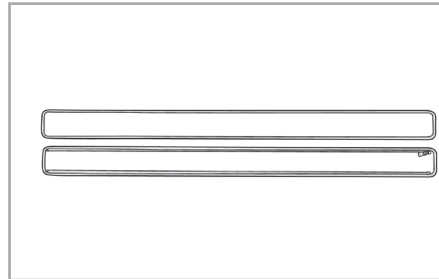

2 x Extension zippers  
for 8-chamber full pants

CPLONGPANTSEXT8

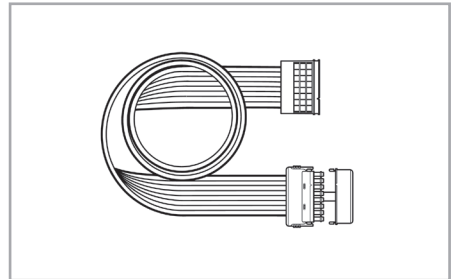


Double splitter (long - 150 cm)  
for 8-chamber devices

CPDOUBLECONNECTOR8L

# Therapeutic pre-programs




## warm-up, prevention

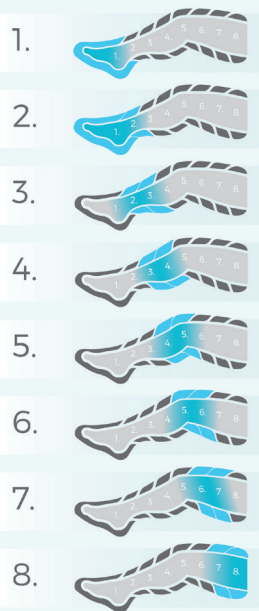


Gradient	0 mmHg
Time	60 min.
Hold	2 s
Interval	2 s

## regeneration, wellness

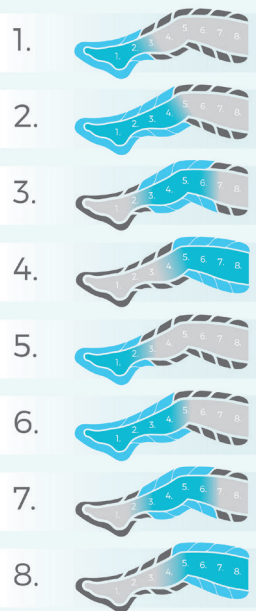


Gradient	4 mmHg
Time	45 min.
Hold	2 s
Interval	2 s

## post-injury therapy

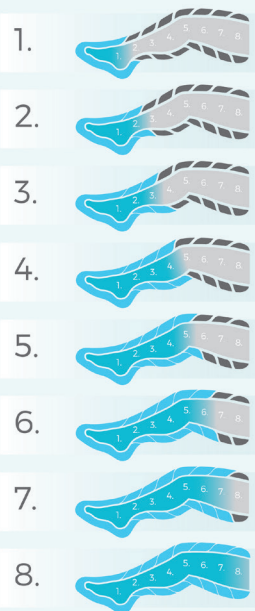


Gradient	4 mmHg
Time	30 min.
Hold	2 s
Interval	2 s

## oedema therapy

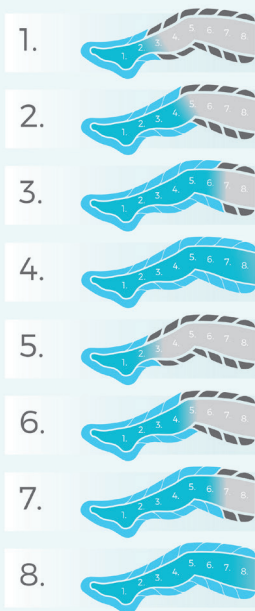

Gradient	1 mmHg
Time	50 min.
Hold	3 s
Interval	3 s

## all-chamber massage

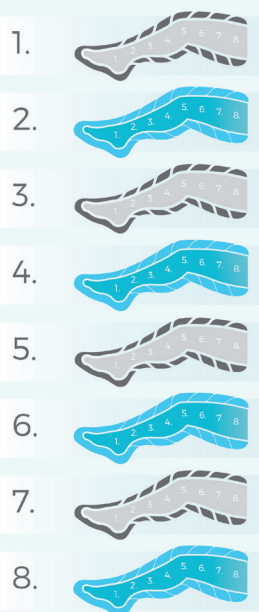

Gradient	0 mmHg
Time	30 min.
Hold	2 s
Interval	3 s

## mixed mode

**Possibility of combining any therapeutic pre-programs and selecting the number of repetitions for each of them.**