

Patient lift

Samssoft ELITE 180



**Reliable,
intuitive and compact,
designed for optimum use**

Chaque jour, nous améliorons **votre vie**

drive

DeVilbiss[®]
HEALTHCARE

Samsoft **ELITE 180**

Home or community use

Ergonomic cross-bar
minimizing the risk
during transfer.

Sturdy and secure
structure.

**Large lifting
amplitude :
from 470 to 1780 mm.
Facilitates ground
pickup.**

Opening between legs :
**897 mm (M version)
991 mm (E version) .**

Ground clearance : 27 mm
to facilitate the passage
of doorways.

Centre locking system
of hoist legs in open
or close position.

Double roller castors, Ø 100 mm.
High quality rolling performance and
strength

**Leg operating pedal
for optimized handling.**

**Large multi-position
ergonomic push handle.**

Magnetic remote control.
Allows you to attach it
anywhere on the device.

**Available electrical
powered leg opening**



Samsoft **ELITE 180**

Fast folding, easy and safe




Comes standard with a mast lock/release trigger for double safety.

Magnetic system to hold and lock in rotation the cross-bar for safe transport.

 **Tool-Free**

Folding in seconds without any disassembly. The cross-bar is inserted into the mast using the remote control. The actuator remains fully protected by the structure. **Can be stored against the bed for space saving.**

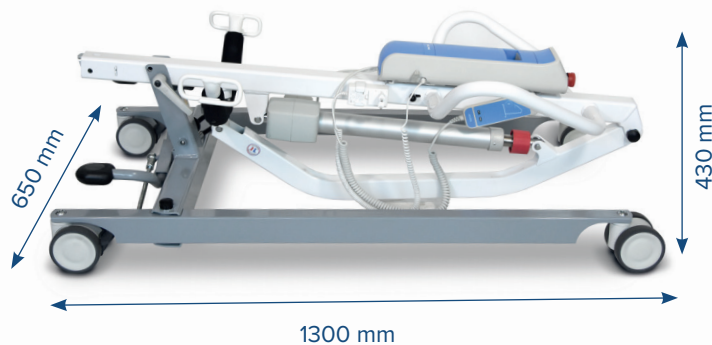

Can also be disassembled into two parts to facilitate its transport.



Power supply box



Emergency button, positioned to prevent accidental activation.

Colour LCD display with many features:
recording of incidents, reminder of maintenance operations, battery charge level.



Operating buttons, an alternative to the remote control.

IPX6 waterproofing: for use in environments subject to water splashes.

Power supply

Lithium-ion battery, lightweight, durable and ultra compact.

Battery charging, 2 possibilities : on base or directly on main socket.

By connecting the unit directly to a wall outlet using the integrated charger **or** remotely by removing the battery and connecting it to the charging dock, to continue using the patient lift with a second battery.



Battery warranty 1 year
Weight : 680 g



Sold as accessory

Security



Standard 4-point aligned cross-bar with curved anti-return hooks for easy, safe hooking.



5-year warranty on structure and actuator*



2-year warranty on box and remote control*



1-year warranty on battery*



By flashing this QR Code, you have access to :

- The user's manual of the device.
- An operating procedure on good predictive maintenance practices.
- User videos and tutorials.


High visibility of safety features

- 1 Emergency lowering
- 2 Emergency button
- 3 Mast locking pin



* Under normal use in accordance with the instructions in the manual. Negligence or handling errors are excluded from the warranty.

Accessories

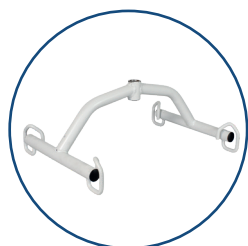

Offset 4-point
cross-bar
Ref. : SR3120700



Protective sleeve
compatible with
both cross-bar versions
Ref. : SR3143900



Charging dock
Ref. : SR3131900



Additional battery
Ref. : SR SR3132000

Slings

Available slings :

- bath netted sling,
- comfort polyester sling,
- netted hammock sling,
- toilet netted hammock sling.



See our documentation
directly online
by scanning this Flash Code

TECHNICAL FEATURES

Reference	 Samsoft ELITE 180	Manual leg opening (M)	SR3110100
		Electrical leg opening (E)	SR3110200
Weight Capacity (Patient & Sling):			180 kg
Minimum lifting height (4-point aligned cross-bar)			470 mm
Maximum lifting height (4-point aligned cross-bar)			1780 mm
Overall width of the base (legs closed)			650 mm
Overall length			1330 mm
Base height (frame)			120 mm
Turning diameter			1430 mm
Ground clearance			(M) : 27 mm (E) : 26 mm
Inner feet spacing			(M) : 1003 mm** (E) : 1120 mm**
Noise level			< 55 dB(A)
Device lifetime			10 years
Patient lift weight (without cross-bar)			(M) : 39,8 kg (E) : 40,2kg

All the photographs are not contractual, and some components may change for manufacturing or improving purposes.

EA 083 09 24 2

** +3/-4

The SAMSOFT ELITE 180 is a battery-powered device for lifting a patient with reduced mobility. It can be operated by a healthcare professional or a trained private individual.

Compliant to ISO 10535:2021



Drive DeVilbiss Healthcare France

Chaussée du Ban la Dame - Parc d'activités Eiffel Energie
ZAC du Ban la Dame - BP 19 - 54390 FROUARD - FRANCE
Tel. +33 (0)3 83 495 495 - www.drivedevilbiss.fr